NEW EDITION 2018

C&C
Carbide & Ceramic

Swiss Dental Diamond
since 1968 by Vanetti SA
www.vanetti.ch

**Utensili in carburo di tungsteno e ceramica • Instruments en carbure de tungstène et céramique
Hartmetall- und Keramikinstrumente • Tungsten carbide and ceramic instruments
Instrumentos en carburo de tungsteno y cerámica • Твердосплавно-вольфрамовые и керамические боры**

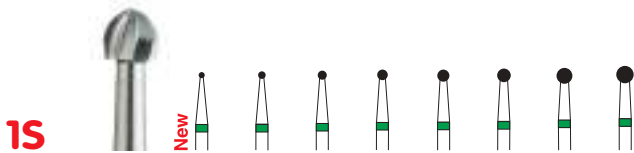
**Gambo • Tige • Shaft
Shank • Mango • Стержень**

iso 313	
FGC - FGS - FGK - FGC - FGC	∅ 1.60 mm
iso 314	
FG	∅ 1.60 mm
iso 316	
FGXL	∅ 1.60 mm
iso 204	
CA - RA - W - CA - CA	∅ 2.35 mm
iso 205	
CAL - RAL - WL - CAL - CAL	∅ 2.35 mm
iso 206	
CAXL - RAXL - WXL - CAXL - CAXL	∅ 2.35 mm
iso 104	
PM - HP - H - PM - PM	∅ 2.35 mm


**Esempio d'ordine • Exemple de commande
Bestellungsbeispiel • Order example • Ejemplo de pedido
Пример заказа**

REF.-No	1	1	1
Fig.-No ISO	001001	001001	001001
ISO Ø 1/10mm	P		
314 FG	21	005	006 008
316 FG XL			
204 RA	21	005	006 008
205 RAL			

P : Prezzi - Prix - Preise - Prices - Precios
Цена по категории : 21 - 22 - 23 ...

REF.-No	1S	1S	1S	1S	1S	1S	1S	1S
Fig.-No ISO	001003	001003	001003	001003	001003	001003	001003	001003
ISO Ø 1/10mm	P							
314 FG	23	008	010	012	014	016	018	021 023
204 RA	23	008	010	012	014	016	018	021 023
205 RAL	24	010	012	014	016	018	021	023

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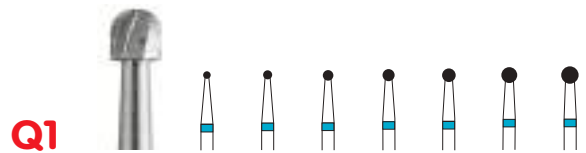GB USER INFORMATION - HYGIENE RECOMMENDATIONS

User informations.

- Use adequate supply of water spray in order not to damage the tooth and contiguous tissues. The water should be distributed over the entire length of the working part. A deflection of the coolant jet can cause heat related damages.
- Additional cooling is required for instruments with a total length exceeding 19 mm and for instruments with a head diameter exceeding 2 mm.
- Immediately remove any instruments that are damaged, bent or do not run concentrically.
- Pressure should be 0,3 - 2N (30 - 200p) for FG instruments and 2 - 5N (200 - 500p) for HP instruments, in other words work with as little pressure as possible.
- The turbine, right-angle and handpiece must be in perfect technical conditions.
- Insert the instruments carefully and without using force. Miniature instruments should be inserted no further than the end of the cylindrical part.
- Avoid canting or levering when grinding.
- Observe the recommended speeds indicated in the table below and on packages.
- Full speed should be reached outside the mouth of the patient. Technicians should run the handpiece at full speed before application to the product on which is being performed.

Hygiene recommendations (disinfection, cleaning, sterilization and storage of the Dia-Tessin instruments).

- The instruments must always be disinfected, cleaned and sterilized before they are used on patients.
- Immediately place used instruments in a special anti-corrosive disinfectant bath for a short time otherwise their colour code may come off. Do not use too aggressive products (hydrochloric acid, hydrogen peroxide,...), they may corrode the instruments.
- Clean the instruments in an ultrasonic bath or under running water (preferably distilled water).
- After cleaning, immediately dry the instruments otherwise they can corrode.
- Before sterilization, place the instruments in stands and trays suitable for sterilization.
- Sterilize the instruments. Autoclave: 18 min. 134°C, 2 bars.
- After sterilization, keep the instruments in dust-free places. Instruments used for surgery should be kept in sealed containers.

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